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## Transaroll Silverboards

EN

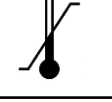
### Patient transfer aids in various designs

Please read the usage instructions carefully before first use!

### Icon legend

	Bed-to bed transfer		Max. weight load		Observe operating instructions
	Warning		Manufacturer		Mark of conformity
	Temperature limitation		Keep away from sunlight		Medical device Designation
	Item number		Batch no./Lot no.		The bridging of free transfer levels must be not exceed 15 cm
	Keep dry		Slip hazard		Latex-free

## 1. Technical data

REF	Item designation / variants	Dimensions	UDI-DI
4500028	Transaroll Silverboard Mini, foldable	95 x 50 cm	(01)04050856034557(11)xx(10)xx
4500029	Transaroll Silverboard Mini, non-foldable	95 x 50 cm	(01)04050856034373(11)xx(10)xx
4500030	Transaroll Silverboard Standard, foldable	180 x 50 cm	(01)04050856034366(11)xx(10)xx
4500033	Transaroll Silverboard Special, foldable	150 x 60 cm	(01)04050856043900(11)xx(10)xx
4500034	Transaroll Silverboard Special, non foldable	150 x 60 cm	(01)04050856043924(11)xx(10)xx

	Material	Colour	Property	Rolling / tensile load	Rolling / tensile load
Transaroll Silverboard	Polyethylene, latex-free	black	radiolucent	200 kg	max. 300 kg* in combination with the one-way sliding film
Cover	PES fabric with PVC coating	silver			
Undercover	PES fabric with silicone coating	blue			

\* Please speak to the responsible medical device consultant if necessary.

### 1.1 Storage and transport

Storage temperature: 10° to 30°

Storage conditions: keep dry, protect from moisture and direct sunlight

Humidity: no specifications

### 1.2 Accessories

Item-No.	Item designation / variants	Dimensions	Material	Colour	UDI-DI
4400200	Disposable sliding film in one piece, can be cut to size individually	60 cm x 100 m	Polyethylene (LDPE)	green	(01)4250355360601(11)xx
4400203	Disposable sliding film in one piece, incl. knife for cutting	60 cm x 100 m	Polyethylene (LDPE)	green	(01)4250355360656(11)xx
4400204	Disposable sliding film on a roll of 60 pieces	53 cm x 180 cm	Polyethylene (LDPE)	green	(01)4250355394729(11)xx

Item No.	Item designation / variants	Dimensions	Material	Colour
4500120	Disposable protective film standard box containing 50 pieces (folded) not antistatic	60 cm x 195 cm	Polyethylene (LDPE)	transparent
4500125N	Disposable protective film standard 100 pcs. per roll, perforated antistatic	56 cm x 195 cm	Polyethylene (LDPE)	transparent
4500126	Disposable protective film 150 pcs. per roll, perforated, antistatic	54 cm x 100 cm	Polyethylene (LDPE)	transparent
4500417	Organiser for roller boards, board storage and dispenser for disposable sliding or protective film in one	70,5 cm x 119 cm x 17 cm	Polyester fabric, coated	silver

---

## 2. Indication / areas of application / intended use

---

No specific clinical picture can be given as an indication because of the various possible applications. The Transaroll Silverboard is suitable for all applications on persons requiring assistance with lateral transfers.

The Silverboard can be used to bridge gaps when two lying surfaces are positioned next to each other (e.g. nursing bed – nursing bed, nursing bed – shower trolley, nursing bed – operating table, stretcher – examination table).

The Transaroll Silverboard is suitable for gentle, pain-free and low-friction lateral transfer of persons whose movement is limited.

---

## 3. Redeployment

---

The Transaroll Silverboards are intended for reuse. They are not a disposable product.

Any accessories used, such as sliding or protective film, are disposable products and must not be re-used, since this may cause material damages and consequently harm the patient's health.

It is essential to conduct a thorough functional check and to clean or disinfect the basic structure before using it again.

---

## 4. Contraindications / risks of use

---

There are no known contraindications.

Users must be trained. If users are not trained or acquainted with the correct handling of the Transaroll Silverboard, the board may be lifted or pushed incorrectly. Movements may also be performed too quickly and abruptly. This might result in damage to the health of the patient and/or user. Observe the application instructions under point 5 and the safety instructions under point 7.

It is the sole responsibility of the user to determine the suitability of the Transatlantic Silverboard for the specific application. Transatlantic shall not be held accountable for any improper use of Transaroll products.

## 5. Use / placement

Position the board to the side, next to the patient.

Position yourself next to the transfer level and push the patient in the desired transfer direction.

Ensure that the brake is on!

To remove the board, reposition the patient in a short-term gentle lateral position.

Tilt the patient to the side gently and slide the board slightly from under the patient.

The board can now be removed from under the patient.

## 6. Service life / life cycle

The expected service life of the Transaroll Silverboard is 5 years if it is used in accordance with the safety instructions. There is no limitation to this period.

A longer or shorter service life is possible depending on the load. Usability may also end prematurely in the event of relevant damage to the patient transport aids.

## 7. Safety instructions / functional check

**WARNING** signifies a possible hazard that could lead to personal injury or material damage.

- Defective products must no longer be used.
- Slip hazard! Ensure that the patient does not slide off the bed, recliner, etc. during transfer or repositioning.
- Risk of accident! Always make sure when using the Silverboard that the bed, recliner or any movable base is stable, i.e. that any brakes on it are locked.
- Ensure that the Silverboard is stable on both sides at the transfer levels. The bridging of free transfer levels with the Silverboard must not exceed 15 cm.
- It is the responsibility of the user/institution to evaluate if multiple users may be needed for the transfer or if additional safety measures must be taken.

- Never leave the patient unsupervised on the Transaroll Silverboard.
- The Transaroll Silverboard must not remain under the patient after transfer.
- Pay attention, particularly when lifting the Transaroll Silverboard for a short time, that the patient is lying securely on the board and cannot slide down.
- Make sure that the transfers are performed by the users in a way that is back-friendly.
- Weight limits must be observed 200 kg, optionally 300 kg when used with the disposable sliding film.

Repositioning products are also subject to a natural ageing process. Inspect the product regularly and carefully. Ensure that the product is in flawless condition and that the material has not been damaged.

Transaroll transfer aids are subject to the highest quality requirements Any serious incidents must be reported to the manufacturer and the competent authority in the Member State where the user and/or patient is based.

---

## **8. Cleaning, care and disinfection**

---

The silver cover must be cleaned with medical disinfectants prior to and after use each time a patient is changed.

### **8.1. Blue undercover**

When cleaning, do not use abrasive or bleaching agents and objects with sharp edges. The use of bleaching agents or cleaning agents that contain alcohol can permanently impair the functional properties of the blue cover. The blue cover can be easily disinfected using oxygen-based disinfectant solutions.

### **8.2. Silver cover**

When cleaning, do not use abrasive or bleaching agents and objects with sharp edges. The use of bleaching agents can permanently impair the functional properties of the cover. The silver cover can be easily disinfected using alcohol-based disinfectant solutions. Please use the disposable protective film for use in hygienically sensitive areas.

### **8.3. Board**

The board itself withstands a lot of wear and tear and can be cleaned using normal household cleaners. Conventional cleaning: Use warm water and a mild cleaning agent for cleaning. If necessary, wipe the board using a damp cloth and then leave it to dry thoroughly.

Medical cleaning: The board can be effortlessly disinfected using an alcohol-based disinfectant solution. Always leave to dry thoroughly. The board is unsuitable for use in an autoclave due to the thermo-sensitive structure of the material.

---

## **9. Product marking /CE**

---

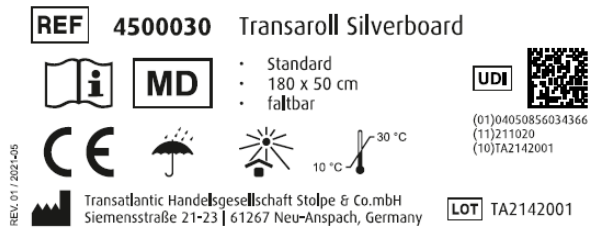
The Transatlantic Silverboard is a Class I medical device and is compliant with EU Regulation 2017/745 of the European Parliament and of the Council dated 05.04.2017 regarding medical devices (MDR).

We are certified as a manufacturer and distributor of medical devices in accordance with DIN EN ISO 9001:2015 as well as DIN EN ISO 13485:2016. Our medical products have, of course, passed the relevant conformity assessment procedure and therefore bear the CE mark.

## Example of product marking

## Example of packaging marking

## 10. Warranty

The warranty period is 3 years. This shall not extend to defects that are attributable to improper use or normal wear and tear. Transatlantic shall not be held liable for accidental or indirect injury or material damage.

## 11. Disposal information

The Transatlantic Silverboard is not subject to any special disposal regulations. It must be disposed of in accordance with national and local legislation.

**Transatlantic Handelsgesellschaft  
Stolpe & Co. mbH**  
Siemensstraße 21-23  
61267 Neu-Anspach, Germany  
Tel.: +49 (0)6081 9430-50  
Fax: +49 (0)6081 9430-80  
info@transat.de  
www.transatlantic.de

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