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Instructions for Use Transatlantic Beasy Boards

EN

Transatlantic Beasy-Boards

Please read these instructions for use carefully before using the Transatlantic Beasy board for the first time!

Symbol legend:

	Warning		Max. weight load		Observe instructions for use
	Item number		Manufacturer		Mark of conformity
	Batch no. / Lot. no.		Keep away from sunlight		Medical product marking
	Slip hazard		Quantity per package / carton		

1. Technical data / product variants

Item No.	Item designation	Dimensions	UDI-DI
4411068	Beasy II	68 x 30 cm	(01)04050856016348(10)xx
4411080	Beasy Glyder	80 x 25,5 cm	(01)04050856016362(10)xx
4411102	Beasy Trans	102 x 30 cm	(01)04050856016386(10)xx

Item No.	Material	Color	Weight	Transfer load
4411068	Polypropylene, polyethylene, polyamide, ABS	light gray	2160 g	180 kg
4411080			2270 g	180 kg
4411102			3400 g	180 kg

1.1 Storage

Storage conditions: keep dry, protect from moisture and direct sunlight

1.2 Accessories

Special transport bags can be requested. We have a variety of patient transfer aids in our product range that can be a helpful addition to the Transatlantic Beasy board selected. If you are interested, please contact your responsible medical product advisor. Our office staff will also be happy to help you.

2. Indication / application areas / intended use

No specific clinical picture can be given as an indication because of the various possible applications. The Transatlantic Beasy board is suitable for all applications to support persons requiring assistance when transferring from one location to another in a sitting position.

The design of the Transatlantic Beasy boards allows for a sliding transfer in clinical as well as home-care settings. Especially the transfer from wheelchair to automobile as well as shifting a patient in bed is assisted by the Transatlantic Beasy board. The Beasy board can also be used independently, particularly for transfer in confined spaces e.g., in a bathroom (after consultation with the person in charge).

3. Multiple use

Transatlantic Beasy boards are intended for multiple use. They are not a disposable product.

Make sure to conduct a thorough functional check (see Section 8) of the product and to clean or disinfect it (see Section 6) before re-use.

4. Contraindications / risks of use

There are no known contraindications.

Users must be trained. If users are not trained in or familiarized with the correct handling of the Transatlantic Beasy board, there is a risk that the board will be raised or pushed incorrectly. There is also a risk that movements will be carried out too quickly and abruptly. This could result in harm to the health of the patient and/or user. Observe the usage instructions in Section 5 and the safety instructions in Section 7.

It is the sole responsibility of the user to determine the suitability of the Transatlantic Beasy board for the specific application. Transatlantic assumes no responsibility for any improper use of Transatlantic Beasy board products.

5. Usage / placement

Example: Transfer from bed to wheelchair / wheelchair to bed

Step 1: Provide for a level transfer. This step is very important. When transferring from bed to wheelchair or from wheelchair to bed, adjust the bed height to the seat height of the wheelchair as much as possible.

Step 2: Remove the armrest and footrest of the wheelchair on the transfer side; fold away the other footrest. Position the wheelchair so that you have maximum freedom of movement. Having sufficient space will help you avoid injuries through contact with surrounding objects.

Note: Always place the wheelchair on the patient's strong side. To check whether the wheelchair is at the correct angle, you can perform a "dry run" by sliding the rotating seat across the board by hand. In this way, possible obstacles or problems with positioning are detected before the patient is moved.

Note: Always consult a competent physician/professional before performing a transfer. Never cross the patient's legs if he/she has an artificial hip or similar problems exist.

Step 3: Position the wheelchair at the correct angle. All Transatlantic Beasy board models are designed to allow the user to move comfortably from one location to another in a sliding process. It is important to plan the transfer path in advance and to position the wheelchair at the correct angle to the bed or other transfer sitting surfaces. An angle of 45° usually works best here.

Step 4: Lock the wheelchair brakes! As with all transfers, make sure the wheelchair brakes are locked.

Step 5: Slide the rotating seat under the patient's upper thigh so that its leading edge is visible. The patient should be asked to lean as far as possible to one side so that the rotating seat can be positioned more easily. As a general rule, this teamwork should last for the entire transfer process. The rotating seat should be placed under the patient's upper thigh, not directly under the buttocks.

Note: Always make sure that both ends of the Transatlantic Beasy board are supported securely. The caregiver lifts the patient's leading leg and places it over the back leg. The patient should be positioned slightly towards the back of the seat. The leading edge of the rotating seat should always be visible before the transfer is made. The caregiver now grasps the rotating seat and presses it down slightly so that it cannot slide away during placement.

Place the legs parallel to each other again and help the patient to sit upright. Make sure that the leading edge of the rotating seat remains visible. If this is not the case, the transfer must be aborted. The use of a gait belt to provide the patient's lower body with additional support is recommended.

The transfer can begin when:

1. The transfer levels are at the same height.
2. The wheelchair is correctly positioned and the brakes are locked.
3. The patient is seated so that the leading edge of the rotating seat is visible.
4. The Transatlantic Beasy board is stably supported at both ends.

The transfer

After the four preparatory steps have been completed, you are now ready for the transfer. Always grasp the patient, as low as possible, either at the hips or buttocks (alternatively, you can use a gait belt). Now place the patient's legs between those of the caregiver and lean the patient slightly forward so that he/she is in a manner "hugged".

The transfer can now be made in a flowing sideways movement or, if necessary, in short sideways movements. Agree among yourselves on the best alternative when performing the transfer.

Ending the transfer

The system glides at the desired speed. Perform the transfer at a slow to medium pace. Take advantage of the transfer momentum, the seat rotation and the slight S-shape of the Beasy board to bring the patient into the correct final position at the end of the transfer.

Note: The Transatlantic Beasy board can be conveniently removed after the transfer is complete by grasping the base of the board and turning it in a wide arc.

For further details, please refer to our website www.transatlantic.de

6. Cleaning, care and disinfection

The Transatlantic Beasy boards should be cleaned before each use. The Beasy board can be cleaned with dishwashing detergent and lukewarm water. You can also use a non-abrasive detergent suitable for plastics.

When required (after use in hygienically critical areas, such as toilets), we recommend particularly thorough cleaning. Our Transatlantic Beasy boards are easily disassembled by removing the track guard pins (with a flat-tip screwdriver and a few light taps with a hammer) and unscrewing the nut that holds the seat to the base. After cleaning, we strongly recommend lightly oiling the front of the board and rubbing in the oil with a soft cloth. The nut should be tightened with your fingers and then loosened until the board can slide freely. The nut should not be tightened firmly! Replace the pins of the track guard using needle-nosed pliers and by lightly tapping with a hammer. Check your Transatlantic Beasy board regularly for signs of cracking or damage. Do not use the Transatlantic Beasy board shows any signs of damage.

7. Service life / life cycle

If the product is used in accordance with the safety instructions, the expected product service life is 5 years. However, the effective service life may vary depending on the frequency and intensity of use as well as the number of cleaning and disinfection operations. Carry out a functional check with the product on a regular basis and be sure to observe the safety instructions (see Section 8).

8. Safety instructions / functional check

WARNING signifies a possible hazard that could lead to personal injury or material damage.

- Defective products must no longer be used.
- Slip hazard! Ensure that the patient does not slide off the wheelchair, recliner etc. during transfer or repositioning.
- Risk of accident! When using the Transatlantic Beasy board, always make sure that the wheelchair, recliner or any movable support is stable, i.e. that any brakes there are locked.
- Ensure that the Transatlantic Beasy board is stably supported on both sides at the transfer levels.
- Make sure that, appropriate to the situation, a sufficient number of users provide for a safe transfer.
- Never leave the patient unsupervised on the Transatlantic Beasy board.
- Make sure that the patient is seated/lying securely on the Transatlantic Beasy board and cannot slide off.
- Make sure that transfers are performed by the users in a way that is back-friendly.
- Weight limits must be observed (180 kg).

Repositioning products are also subject to a natural aging process. Inspect the product regularly and carefully. Ensure that the product is in flawless condition and that the material has not been damaged.

Transatlantic transfer aids are subject to the highest quality standards. Any serious incidents must be reported to the manufacturer and the competent authority in the Member State in which the user and/or patient resides.

9. Product marking /CE

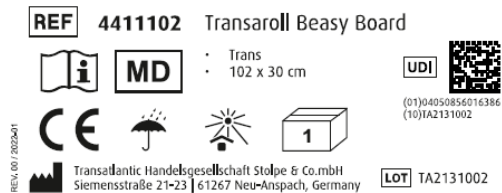
The Transatlantic Beasy board is a Class I medical product and complies with EU Regulation 2017/745 of the European Parliament and of the Council of 04/05/2017 for medical products, also referred to as Medical Device Regulation (MDR).

We have been assessed and are certified as a manufacturer and distributor of medical products in accordance with DIN EN ISO 9001 and DIN EN ISO 13485. Our medical products have, of course, undergone the relevant conformity assessment procedure and therefore bear the CE mark.

Example marking of product:

Example marking of packaging:

10. Warranty

The warranty period is 2 years. This does not extend to defects that are attributable to improper use or normal wear and tear.

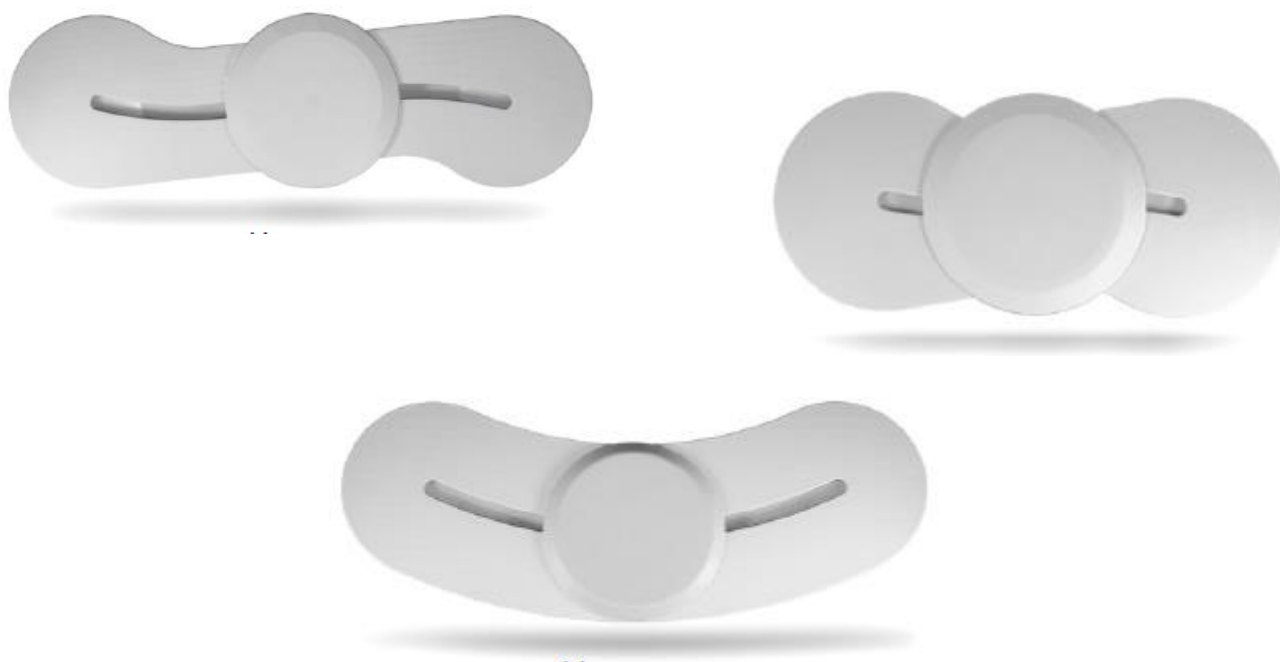
Transatlantic assumes no liability for accidental or indirect injury or for material damage.

11. Disposal information

The Transatlantic Beasy board is not subject to any special disposal regulation. It must be disposed of in accordance with national and local regulations.

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