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Instruction for use Transaroll Klipp-Klapp Board

EN

Please read the usage instructions carefully before first use!

Icon legend

	Mark of conformity		max. weight load		Follow instruction for use
	Warning		Manufacturer		Temperature limitation
	Keep dry		Keep away from sunlight		Marking Medical Device
	Item number		Batch / Lot no.		Slip hazard
	The bridging of free transfer levels must be not exceed 15 cm				Marking UDI

1. Technical Data

REF	Article description	Dimension	UDI-DI
4400103	Transaroll Klipp-Klapp Board	75 x 45 cm, foldable to 25 x 45 cm	(01)04050856038845(11)xx(10)xx

	Material	Colour	Rolling- tensile- and weight load
Base body Klipp-Klapp Board - Ambu, 75 x 45 cm	High-density polypropylene (HDPE)	transparent	max. 180 kg
Replacement cover Klipp-Klapp Board - Ambu, 75 x 45 cm	Gliding fabric with coated cover, welded bond, seamed	grey	

1.1 Storage and transport

Storage temperature:

5° bis 25° C

Storage conditions:

keep dry, protect from moisture and direct sunlight

Humidity:

no specifications

2. Indications / Areas of Application / Intended Use

Due to the various possible applications, no specific clinical picture can be used as an indication. The Transaroll Klipp-Klapp Board is suitable for all applications on people with support needs during lateral transfer.

The Transaroll Klipp-Klapp Board is suitable for gentle and low-friction lateral transfer of recumbent patients during rescue operations.

3. Redeployment

The Transaroll Klipp-Klapp Boards are intended for reuse. It is not a disposable product. Before using the Transaroll Klipp-Klapp Board again, be sure to carry out a thorough functional check. The Transaroll Klipp-Klapp Board must be cleaned with medical disinfectants before and after each patient use.

4. Contraindications / Risks of Use

There are no known contraindications.

Users must be trained. If the users are not trained or familiar with the correct handling of the Transaroll

clip-fold board, the board may be lifted or pushed incorrectly. Furthermore, movements can be performed too quickly and jerkily. This could lead to harm to the health of the patient and/or user. Observe the instructions for use under point 5 and the safety instructions under point 7. Determining the suitability for use of the Transaroll Klipp-Klapp Board is the sole responsibility of the user. Transatlantic accepts no responsibility for improper use of Transaroll products.

5. Application/ Placement

The transfer is also possible with a distance of up to 15 cm between individual levels. For safety reasons, the Transaroll Klipp-Klapp Board should be used by two people and, if necessary, with a sheet as a draw sheet. Put the patient in the smallest possible side position and push the board slightly under the patient.

for a short time and then push the Transaroll Klipp-Klapp Board slightly sideways under the patient. Place the patient in a supine position on the Klipp-Klapp Board. Stand next to the transfer level and push the patient onto the Klipp-Klapp Board in the transfer direction.

When removing the Klipp-Klapp Board, bring the patient back into a slightly side position for a short time and then remove the Klipp-Klapp Board from under the patient.

6. Service life / life cycle

If the Transaroll Klipp-Klapp Board is used in accordance with the safety regulations, the expected useful life is 5 years. There is no limitation to this period.

Depending on the stress, a longer or shorter service life is possible. In the event of relevant damage to the patient transport aids, premature non-usability is also possible.

7. Safety instructions / functional check

WARNING signifies a possible hazard that could lead to personal injury or material damage.

- Defective products must no longer be used.
- Slip hazard! Ensure that the patient does not slide off the bed, recliner, etc. during transfer or repositioning.
- Risk of accident! Always make sure when using the Transaroll Klipp-Klapp Board that the bed, recliner or any movable base is stable, i.e. that any brakes on it are locked.
- Ensure that the Transaroll Klipp-Klapp Board is stable on both sides at the transfer levels. The bridging of free transfer levels with the Transaroll Klipp-Klapp Board must not exceed 15 cm.
- It is the responsibility of the user/institution to evaluate if multiple users may be needed for the transfer or if additional safety measures must be taken.
- Never leave the patient unsupervised on the Transaroll Klipp-Klapp Board .
- The Transaroll Klipp-Klapp Board may only remain under the patient for a short time after the transfer.
- Pay attention, particularly when lifting the Transaroll Klipp-Klapp Board for a short time, that the patient is lying securely on the board and cannot slide down.
- Make sure that the transfers are performed by the users in a way that is back-friendly.
- Weight limits must be observed: Rolling-, tensile- and weight load max. 180 kg

Repositioning products are also subject to a natural ageing process. Inspect the product regularly and carefully. Ensure that the product is in flawless condition and that the material has not been damaged.

Transaroll transfer aids are subject to the highest quality requirements Any serious incidents must be reported to the manufacturer and the competent authority in the Member State where the user and/or patient is based.

8. Cleaning, care and disinfection

The Transaroll Klipp-Klapp Board must be cleaned with medical disinfectants before and after each patient use.

8.1. Grey Cover

Do not use abrasive or bleaching agents or sharp-edged objects for cleaning. The use of bleaching agents or cleaning agents containing alcohol can permanently impair the functional properties of the

grey cover. The grey cover can be easily disinfected with oxygen-based disinfecting solutions.

8.2. Board

The board itself is very resistant and can be cleaned with normal household cleaners.

Conventional cleaning: Use warm water and a mild detergent for cleaning. If necessary, wipe the board with a damp cloth and then allow it to dry thoroughly.

Medical cleaning: The board can be easily disinfected with an alcohol-based disinfectant solution. Then be sure to let it dry completely. The board itself is not suitable for use in an autoclave due to the thermo-sensitive structure of the material.

9. Product labelling /CE

The Transaroll Klipp-Klapp Board is a Class I medical device and complies with EU Regulation 2017/745 of the European Parliament and of the Council of 04/05/2017 for medical devices (MDR).

As a manufacturer and distributor of medical devices, we are certified according to DIN EN ISO13485.

Example of product labelling:

10. Warranty

The warranty covers a period of 1 year. This does not cover defects that are due to improper use or normal wear and tear.

Transatlantic is not liable for accidental or indirect injury or property damage.

11. Notice

Please report any serious incidents related to this product to Transatlantic Handels-Gesellschaft (qm@transatlantic.de) and the Competent Authority in Europe or the Competent Regulatory Authority in the country where the incident occurred.

12. Disposal information

There are no special disposal regulations for the Transaroll Klipp-Klapp Board. It must be disposed of in accordance with national and local regulations.

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REV.: 12 / © Transatlantic - Art-Nr. / Item-No. / Numéro d'article: : Z100047