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EN

Transaroll Klipp-Klapp Board

Please read the instructions for use carefully before the first use!

Icon legend

	Bed-to-bed transfer		max. weight load		Follow the instruction for use
	Warning		Manufacturer		Mark of conformity
	Temperature limitation		Keep away from sunlighth		Medical device label
	Item number		Batch no./Lot no.		The bridging of open transfer levels must not exceed 15 cm
	Keep dry		Slip hazard		Latex-free

1. Technical data

REF	Item designation / variants	Dimensions	UDI-DI
4400103	Transaroll Klipp-Klapp Board	75 x 45 cm 2x foldable to 25 x 45 cm	(01)04050856038845(11)xx(10)xx

	Material	Colour	Rolling / tensile load
Base body for Transaroll Klipp-Klapp Board - Ambu, 75 x 45 cm	High-density polypropylene (HDPE)	transparent	max. 180 kg
Replacement roll-up cloth for Transaroll Klipp-Klapp Board - Ambu, 75 x 45 cm	Gliding fabric with coated cover, welded bond, seamed	grey	

1.1 Storage and transport

Storage temperature: 10° to 30° Celsius

Storage conditions: dry, protect from moisture and direct sunlight

Relative humidity: no information

2. Indications / Areas of Application / Intended Use

Due to the various possible applications, no specific clinical picture can be used as an indication. The Transaroll Klipp-Klapp Board is suitable for all applications on people with support needs during lateral transfer.

The Transaroll Klipp-Klapp Board is suitable for gentle and low-friction lateral transfer of recumbent patients during rescue operations.

3. Redeployment

The Transaroll Klipp-Klapp Boards are intended for reuse. It is not a disposable product. Before using the Transaroll Klipp-Klapp Board again, be sure to carry out a thorough functional check. The Transaroll Klipp-Klapp Board must be cleaned with medical disinfectants before and after each patient use.

4. Contraindications / Risks of Use

There are no known contraindications.

Users must be trained. If the users are not trained or familiar with the correct handling of the Transaroll clip-fold board, the board may be lifted or pushed incorrectly. Furthermore, movements can be performed too quickly and jerkily. This could lead to harm to the health of the patient and/or user. Observe the instructions for use under point 5 and the safety instructions under point 7.

Determining the suitability for use of the Transaroll Klipp-Klapp Board is the sole responsibility of the user. Transatlantic accepts no responsibility for improper use of Transaroll products.

5. Application/ Placement

The transfer is also possible with a distance of up to 15 cm between individual levels. For safety reasons, the Transaroll Klipp-Klapp Board should be used by two people and, if necessary, with a sheet as a draw sheet. Put the patient in the smallest possible side position and push the board slightly under the patient.

for a short time and then push the Transaroll Klipp-Klapp Board slightly sideways under the patient. Place the patient in a supine position on the Klipp-Klapp Board. With the draw sheet, pull the patient over and onto the Klipp-Klapp Board in the direction of transfer. Without the draw sheet, stand next to the transfer level and push the patient onto the Klipp-Klapp Board in the transfer direction.

When removing the Klipp-Klapp Board, bring the person back into a slightly side position for a short time and then remove the Klipp-Klapp Board from under the patient.

6. Service life / life cycle

If the Transaroll Klipp-Klapp Board is used in accordance with the safety regulations, the expected useful life is 5 years. There is no limitation to this period.

Depending on the stress, a longer or shorter service life is possible. In the event of relevant damage to the patient transport aids, premature non-usability is also possible.

7. Safety instructions / functional check

WARNING signifies a possible hazard that could lead to personal injury or material damage.

- Defective products must no longer be used.
- Risk of slipping! Make sure that the patient does not slide off the bed, couch, etc. when repositioning or transferring.
- Risk of accident! When using the Transaroll Klipp-Klapp Board, always make sure that the bed, the couch or any movable support is stable, i.e., that the brakes on it are locked
- Make sure that the Transaroll Klipp-Klapp Board is stable on both sides of the transfer levels. The bridging of open transfer levels with the Transaroll Klipp-Klapp Board must not exceed 15 cm.
- It is the responsibility of the user/facility to assess whether multiple users may be required for the transfer or if additional safety measures need to be taken.
- Never leave the patient unattended on the Transaroll Klipp-Klapp Board.
- The Transaroll Klipp-Klapp Board must not remain under the patient after the transfer.
- Make sure, especially when lifting the Transaroll Klipp-Klapp Board for a short time, that the patient lies securely on the board and cannot slide down.
- Make sure that the transfers are carried out in a way that is best for the backs of the patient and the users.
- Weight limits must be observed (max. 180 kg)

Repositioning products are also subject to a natural ageing process. Check the product regularly and carefully. Make sure that the product is in perfect condition and that the material is not damaged.

Transaroll transfer aids are subject to the highest quality requirements. Serious incidents must be reported to the manufacturer and the competent authority of the member country or state in which the user and/or patient resides.

8. Cleaning, care and disinfection

The Transaroll Klipp-Klapp Board must be cleaned with medical disinfectants before and after each patient use.

8.1. Grey Cover Do not use abrasive or bleaching agents or sharp-edged objects for cleaning. The use of bleaching agents or cleaning agents containing alcohol can permanently impair the functional properties of the grey cover. The grey cover can be easily disinfected with oxygen-based disinfecting solutions.

8.2. Board The board itself is very resistant and can be cleaned with normal household cleaners. Conventional cleaning: Use warm water and a mild detergent for cleaning. If necessary, wipe the board with a damp cloth and then allow it to dry thoroughly.

Medical cleaning: The board can be easily disinfected with an alcohol-based disinfectant solution. Then be sure to let it dry completely. The board itself is not suitable for use in an autoclave due to the thermo-sensitive structure of the material.

9. Product labelling /CE

The Transaroll Klipp-Klapp Board is a Class I medical device and complies with EU Regulation 2017/745 of the European Parliament and of the Council of 04/05/2017 for medical devices (MDR).

As a manufacturer and distributor of medical devices, we are certified according to DIN EN ISO 9001 as well as DIN EN ISO 13485. Of course, our medical devices have undergone the relevant conformity assessment procedure and thus bear the CE mark.

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© Transatlantic REV. 0 / 2019-05

10. Warranty

The warranty covers a period of 1 year. This does not cover defects that are due to improper use or normal wear and tear.

Transatlantic is not liable for accidental or indirect injury or property damage.

11. Disposal information

There are no special disposal regulations for the Transaroll Klipp-Klapp Board. It must be disposed of in accordance with national and local regulations.

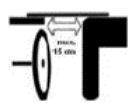
**Transatlantic Handelsgesellschaft
Stolpe & Co. mbH**
Siemensstraße 21-23
61267 Neu-Anspach, Germany
Tel.: +49 (0)6081 9430-50
Fax: +49 (0)6081 9430-80
info@transat.de www.transatlantic.de

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