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Please read the usage instructions carefully before first use!

1. Technical Data

REF	Article description / variants	Dimensions	Weight	UDI-DI
4400114	Transaroll Professionalboard Ambu, non-foldable	80 x 40 cm	2 kg	(01)04050856015501 (11)xx(10)xx
4400420	Transaroll Professionalboard Slim, non-foldable	120 x 40 cm	1,52 kg	(01)04050856038555 (11)xx(10)xx
4400400	Transaroll Professionalboard Mini, non-foldable	82 x 50 cm	2 kg	(01)04050856034014 (11)xx(10)xx
4401100	Transaroll Professionalboard Standard, foldable	180 x 50 cm	3 kg	(01)04050856043191 (11)xx(10)xx
4401150	Transaroll Professionalboard Standard, non-foldable	180 x 50 cm	7,4 kg	(01)04050856036841 (11)xx(10)xx

	Material	Colour	Property	Rolling- tensile- and weight load
Board 4400420 4401100	2-layer PE foam, reinforced with glass fabric in the middle	white	radiolucent	max. 200 kg
Board	2-layer PE foam, reinforced with woodcare in the middle	white		
Cover	PES fabric with plastic coating, with antifungal finish and dirt-repellent protective varnish.	silver		
Undercover	PES fabric with silicone coating	blue		

1.1 Storage and transport

Storage temperature: 5° to 35°

Storage conditions: keep dry, protect from moisture and direct sunlight

Humidity: no specifications

1.2 Accessories

Item-No.	Article description / variants	Dimensions	Material	Colour	UDI-DI
4400200	Disposable sliding film in one piece, can be cut to size individually	60 cm x 100 m	Polyethylene (LDPE)	green	(01)4250355360601 (11)xx
4400203	Disposable sliding film in one piece, incl. knife for cutting	60 cm x 100 m	Polyethylene (LDPE)	green	(01)4250355360656 (11)xx
4400204	Disposable sliding film on a roll of 60 pieces	53 cm x 180 cm	Polyethylene (LDPE)	green	(01)4250355394729 (11)xx

Item No.	Article description / variants	Dimensions	Material	Colour
4500120	Disposable protective film standard box containing 50 pieces (folded) not antistatic	60 cm x 195 cm	Polyethylene (LDPE)	transparent
4500125N	Disposable protective film standard 100 pcs. per roll, perforated antistatic	56 cm x 195 cm	Polyethylene (LDPE)	transparent
4500126	Disposable protective film 150 pcs. per roll, perforated, antistatic	54 cm x 100 cm	Polyethylene (LDPE)	transparent
4400088	Dispenser for disposable protective film for wall mounting	55 x 14 x 15 cm	Stainless steel, powder-coated	white
4400086	Mobile hook for hanging rollboards on smooth surfaces	Ø 120 mm	Plastic and powder-coated stainless steel	white

2. Indication / areas of application / intended use

No specific clinical picture can be given as an indication because of the various possible applications. The Transaroll Professionalboard is suitable for all applications on persons requiring assistance with lateral transfers. The Transaroll Professionalboard can be used to bridge gaps when two lying surfaces are positioned next to each other (e.g. nursing bed – nursing bed, nursing bed – shower trolley, nursing bed – operating table, stretcher – examination table). The Transaroll Professionalboard is suitable for gentle, pain-free and low-friction lateral transfer of persons whose movement is limited.

3. Redeployment

The Transaroll Professionalboards are intended for reuse. They are not a disposable product. Any accessories used, such as sliding or protective film, are disposable products and must not be re-used, since this may cause material damages and consequently harm the patient's health. It is essential to conduct a thorough functional check and to clean or disinfect the basic structure before using it again.

4. Contraindications / risks of use

There are no known contraindications. Users must be trained. If users are not trained or acquainted with the correct handling of the Transaroll Professionalboard, the board may be lifted or pushed incorrectly. Movements may also be performed too quickly and abruptly. This might result in damage to the health of the patient and/or user. Observe the application instructions under point 5 and the safety instructions under point 7. It is the sole responsibility of the user to determine the suitability of the Transaroll Professionalboard for the specific application. Transatlantic shall not be held accountable for any improper use of Transaroll products.

5. Use / placement

Position the board to the side, next to the patient.

Ensure that the brake is on!

Tilt the patient to the side gently and slide the board slightly from under the patient.

Position yourself next to the transfer level and push the patient in the desired transfer direction.

To remove the board, reposition the patient in a short-term gentle lateral position.

The board can now be removed from under the patient.

6. Service life / life cycle

The expected service life of the Transaroll Professionalboard is 5 years if it is used in accordance with the safety instructions. There is no limitation to this period. A longer or shorter service life is possible depending on the load. Usability may also end prematurely in the event of relevant damage to the patient transport aids.

7. Safety instructions / functional check

WARNING signifies a possible hazard that could lead to personal injury or material damage.

- Defective products must no longer be used.
- Slip hazard! Ensure that the patient does not slide off the bed, recliner, etc. during transfer or repositioning. Professionalboard
- Risk of accident! Always make sure when using the Professionalboard that the bed, recliner or any movable base is stable, i.e. that any brakes on it are locked.
- Ensure that the Professionalboard is stable on both sides at the transfer levels. The bridging of free transfer levels with the Professionalboard must not exceed 15 cm.
- It is the responsibility of the user/institution to evaluate if multiple users may be needed for the transfer or if additional safety measures must be taken.
- Never leave the patient unsupervised on the Transaroll Professionalboard.
- The Transaroll Professionalboard may only remain under the patient for a short time after the transfer.
- Pay attention, particularly when lifting the Transaroll Professionalboard for a short time, that the patient is lying securely on the board and cannot slide down.
- Make sure that the transfers are performed by the users in a way that is back-friendly.
- Weight limits must be observed: Rolling-, tensile- and weight load max. 200 kg

Repositioning products are also subject to a natural ageing process. Inspect the product regularly and carefully. Ensure that the product is in flawless condition and that the material has not been damaged.

Transaroll transfer aids are subject to the highest quality requirements Any serious incidents must be reported to the manufacturer and the competent authority in the Member State where the user and/or patient is based.

8. Cleaning, care and disinfection

The silver cover must be cleaned with medical disinfectants prior to and after use each time a patient is changed.

8.1. Blue undercover

When cleaning, do not use abrasive or bleaching agents and objects with sharp edges. The use of bleaching agents or cleaning agents that contain alcohol can permanently impair the functional properties of the blue cover. The blue cover can be easily disinfected using oxygen-based disinfectant solutions.

8.2. Silver cover

When cleaning, do not use abrasive or bleaching agents and objects with sharp edges. The use of bleaching agents can permanently impair the functional properties of the cover. The silver cover can be easily disinfected using alcohol-based disinfectant solutions. Please use the disposable protective film for use in hygienically sensitive areas.

9. Product labelling /CE

The Transaroll Professionalboard is a Class I medical device and is compliant with EU Regulation 2017/745 of the European Parliament and of the Council dated 05.04.2017 regarding medical devices (MDR).

We are certified as a manufacturer and distributor of medical devices in accordance with DIN EN ISO 13485. Our medical products have, of course, passed the relevant conformity assessment procedure and therefore bear the CE mark.

Example of product marking

Example of product marking for item 4400114

Icon legend on page 15

10. Warranty

The warranty covers a period of 3 years. This does not cover defects caused by improper use, normal wear and tear, or improper cleaning or disinfection. Transatlantic shall not be held liable for accidental or indirect injury or material damage.

11. Disposable information

The Transaroll Professionalboard is not subject to any special disposal regulations. It must be disposed of in accordance with national and local legislation.

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REV.: 09 / © Transatlantic - Art.-Nr. / Item-No. / Numéro d'article: : Z100032