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# Instruction for use Transaroll Smartboards

EN

## Transaroll Smartboards – Ambu and Standard

Please read the usage instructions carefully before first use!

### Icon legend

	Bed-to-bed transfer		max. weight load		Observe operating instruction
	Warning		Manufacturer		Mark of conformity
	Temperature limitation		Keep away from sunlight		Medical device designation
	Item number		Batch no./Lot no.		The bridging of free transfer levels must be not exceed 15 cm
	Keep dry		Slip hazard		Latex-free

## 1. Technical data

REF	Item designation / variants	Dimensions	UDI-DI
4400101	Transaroll Smartboard - Ambu, foldable	86 x 40 cm	(01)04050856035516(11)xx(10)xx
4400102	Transaroll Smartboard- Standard, non-foldable	180 x 50 cm	(01)04050856035622(11)xx(10)xx

	Material	Colour	Rolling / tensile load
Basic body Transaroll Smartboard - Ambu	Polypropylene wrapped with PE-foam	white	max. 200 kg
Basic body Transaroll Smartboard - Standard	MDF-wood core wrapped with PE-foam	White	
Cover	PES fabric with PVC coating	blue	

- 1.1 Storage temperature: 10° to 30°  
Storage conditions: keep dry, protect from moisture and direct sunlight  
Humidity: no specifications

### 1.2 Accessories

Item-No.	Item designation / variant	Dimensions	Material	Colour
4500125N	Disposable protective film standard 100 pcs. per roll, perforated antistatic	56 cm x 195 cm	Polyethylene (LDPE)	transparent
4500126	Disposable protective film 150 pcs. per roll, perforated, antistatic	54 cm x 100 cm	Polyethylene (LDPE)	transparent

## 2. Indication / areas of application / intended use

No specific clinical picture can be given as an indication because of the various possible applications. The Transaroll Smartboard is suitable for all applications on persons requiring assistance with lateral transfers.

The Transaroll Smartboard can be used to bridge gaps when two lying surfaces are positioned next to each other (e.g. nursing bed – nursing bed, nursing bed – shower trolley, nursing bed – operating table, stretcher – examination table).

The Transaroll Smartboard is suitable for gentle, pain-free and low-friction lateral transfer of persons whose movement is limited.

### 3. Redeployment

The Transaroll Smartboard are intended for reuse. They are not a disposable product. Any accessories used, such as sliding or protective film, are disposable products and must not be re-used, since this may cause material damages and consequently harm the patient's health. It is essential to conduct a thorough functional check and to clean or disinfect the basic structure before using it again.

### 4. Contraindications / risks of use

There are no known contraindications. Users must be trained. If users are not trained or acquainted with the correct handling of the Transaroll Smartboard, the board may be lifted or pushed incorrectly. Movements may also be performed too quickly and abruptly. This might result in damage to the health of the patient and/or user. Observe the application instructions under point 5 and the safety instructions under point 7. It is the sole responsibility of the user to determine the suitability of the Transaroll Smartboard for the specific application. Transatlantic shall not be held accountable for any improper use of Transaroll products.

### 5. Use / placement

Position the board to the side, next to the patient.

Position yourself next to the transfer level and push the patient in the desired transfer direction.

Ensure that the brake is on!

To remove the board, reposition the patient in a short-term gentle lateral position.

Tilt the patient to the side gently and slide the board slightly from under the patient.

The board can now be removed from under the patient.

---

## 6. Service life / life cycle

---

The expected service life of the Transaroll Smartboard is 5 years if it is used in accordance with the safety instructions. There is no limitation to this period.

A longer or shorter service life is possible depending on the load. Usability may also end prematurely in the event of relevant damage to the patient transport aids.

---

## 7. Safety instructions / functional check

---

**WARNING** signifies a possible hazard that could lead to personal injury or material damage.

- Defective products must no longer be used.
- Slip hazard! Ensure that the patient does not slide off the bed, recliner, etc. during transfer or repositioning.
- Risk of accident! Always make sure when using the Transaroll Smartboard that the bed, recliner or any movable base is stable, i.e. that any brakes on it are locked.
- Ensure that the Transaroll Smartboard is stable on both sides at the transfer levels. The bridging of free transfer levels with the Transaroll Smartboard must not exceed 15 cm.
- It is the responsibility of the user/institution to evaluate if multiple users may be needed for the transfer or if additional safety measures must be taken.
- Never leave the patient unsupervised on the Transaroll Smartboard.
- The Transaroll Smartboard must not remain under the patient after transfer.
- Pay attention, particularly when lifting the Transaroll Smartboard for a short time, that the patient is lying securely on the board and cannot slide down.
- Make sure that the transfers are performed by the users in a way that is back-friendly.
- Weight limits must be observed (max. 200 kg).

Repositioning products are also subject to a natural ageing process. Inspect the product regularly and carefully. Ensure that the product is in flawless condition and that the material has not been damaged.

Transaroll transfer aids are subject to the highest quality requirements any serious incidents must be reported to the manufacturer and the competent authority in the Member State where the user and/or patient is based.

## 8. Cleaning, care and disinfection

The Transaroll Smartboard must be cleaned with medical disinfectants prior to and after use each time a patient is changed.

### 8.1. Blue Cover

When cleaning, do not use abrasive or bleaching agents and objects with sharp edges. The use of bleaching agents or cleaning agents that contain alcohol can permanently impair the functional properties of the blue cover. The blue cover can be easily disinfected using oxygen-based disinfectant solutions.

### 8.2. Basic body

The board itself withstands a lot of wear and tear and can be cleaned using normal household cleaners. Conventional cleaning: Use warm water and a mild cleaning agent for cleaning. If necessary, wipe the board using a damp cloth and then leave it to dry thoroughly.

Medical cleaning: The board can be effortlessly disinfected using an alcohol-based disinfectant solution. Always leave to dry thoroughly. The board is unsuitable for use in an autoclave due to the thermo-sensitive structure of the material.


For use in hygienically sensitive areas, please use the disposable protective film.

## 9. Product marking /CE


The Transaroll Smartboard is a Class I medical device and is compliant with EU Regulation 2017/745 of the European Parliament and of the Council dated 05.04.2017 regarding medical devices (MDR).

We are certified as a manufacturer and distributor of medical devices in accordance with DIN EN ISO 9001 as well as DIN EN ISO 13485. Our medical products have, of course, passed the relevant conformity assessment procedure and therefore bear the CE mark.

### Example of product marking: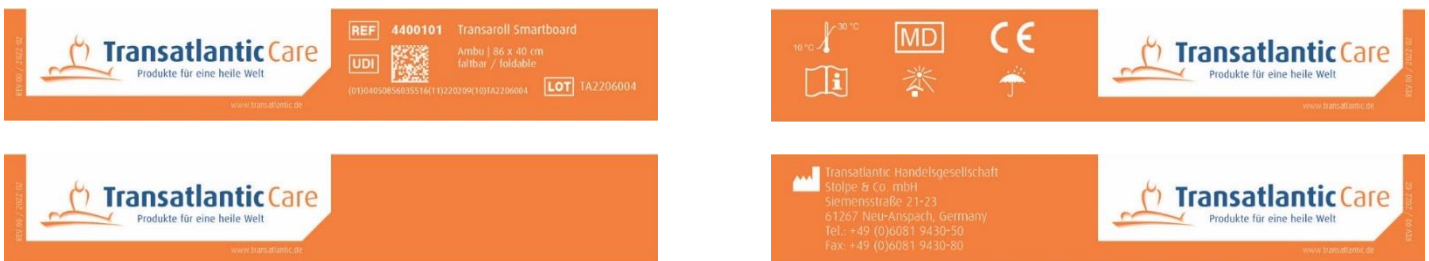

### Example of packaging marking:

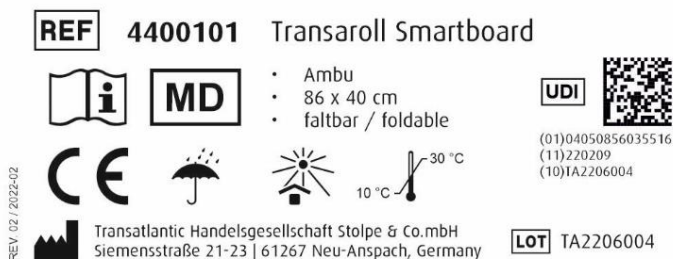---

## 10. Warranty

---

The warranty period is 3 years. This shall not extend to defects that are attributable to improper use or normal wear and tear. Transatlantic shall not be held liable for accidental or indirect injury or material damage.

---

## 11. Disposal information

---

The Transaroll Smartboard is not subject to any special disposal regulations. It must be disposed of in accordance with national and local legislation

**Transatlantic Handelsgesellschaft  
Stolpe & Co. mbH**  
Siemensstraße 21-23  
61267 Neu-Anspach, Germany  
Tel.: +49 (0)6081 9430-50  
Fax: +49 (0)6081 9430-80  
info@transat.de www.transatlantic.de