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Please read the usage instructions carefully before first use!

1. Technical Data

REF	Article description	Dimension	UDI-DI
4500026	Transaroll Hygieneboard Standard, foldable		(01)04050856035493 (11)xxx (10)xxx
	consisting of: - Core, foldable (Item no. 4500412) - Disposable Gliding Cover (Item no. 4400204)	180 x 50 cm 53 x 180 cm	N/A (01)04250355394729 (11)xxx (10)xxx
4500035	Transaroll Hygieneboard Mini, non-foldable		(01)04050856042149 (11)xxx (10)xxx
	consisting of: - Core, non-foldable (Item no. 4500402) - Disposable Gliding Cover (Item no. 4400203)	95 x 50 cm 60 cm x 100 m	N/A (01)04250355364609 (11)xxx (10)xxx
4500036	Transaroll Hygieneboard Mini, foldable		(01)04050856042156 (11)xxx (10)xxx
	consisting of: - Core, foldable (Item no. (Item no. 4500413) - Disposable Gliding Cover (Item no. 4400203)	95 x 50 cm 60 cm x 100 m	N/A (01)04250355364609 (11)xxx (10)xxx

Article description	Material	Colour	Weight	Weight load
Core, Standard	Polyethylene, latex-free	black	1.500 gr.	max. 300 kg
Core, Mini	Polyethylene, latex-free	black	900 gr.	
Disposable Gliding Cover	Polyethylene (LDPE), latex-free	green	6.000 gr	

- 1.1 Packaging: 2 boxes
Scoop of delivery: 1 Core with instruction for use
1 roll of Disposable Gliding Cover with instruction for use
- 1.2. Storage temperature
Core (Board): 5° bis 35° C
Disposable Gliding Cover: 10° bis 25°C
- 1.3. Storage conditions: keep dry, keep away from sunlight
- 1.4. Humidity:
Core (Board): Protect from moisture
Disposable Gliding Cover: 50-75 %

Item-No.	Article description	Dimensions	Material	Colour	UDI-DI
4400200	Disposable gliding cover in one piece, can be cut to size individually	60 cm x 100 m	Polyethylene (LDPE)	green	(01)04250355364609 (11)xxx (10)xxx
4400203	Disposable gliding cover in one piece, incl. knife for cutting	60 cm x 100 m	Polyethylene (LDPE)	green	(01)04250355364609 (11)xxx (10)xxx
4400204	Disposable gliding cover on a roll of 60 pieces	53 cm x 180 cm	Polyethylene (LDPE)	green	(01)04250355394729 (11)xxx (10)xxx

Item No.	Article description	Dimensions	Material	Colour
4400088	Dispenser for disposable protective film for wall mounting	55 x 14 x 15 cm	Stainless steel, powder-coated	white
4400086	Mobile hook for hanging rollboards on smooth surfaces	Ø 120 mm	Plastic and powder-coated stainless steel	white

2. Indication / areas of application / intended use

No specific clinical picture can be given as an indication because of the various possible applications. The Transaroll Hygieneboards is suitable for all applications on persons requiring assistance with lateral transfers. The Transaroll Hygieneboards can be used to bridge gaps when two lying surfaces are positioned next to each other (e.g. nursing bed – nursing bed, nursing bed – shower trolley, nursing bed – operating table, stretcher – examination table). The Transaroll Hygieneboards is suitable for gentle, pain-free and low-friction lateral transfer of persons whose movement is limited.

3. Redeployment

The Transaroll Hygieneboards are intended for reuse. They are not a disposable product. The gliding cover is a disposable product and must not be reprocessed. Reprocessing (washing or disinfecting) the foil can lead to material damage and thus to damage to the patient's health. It is essential to conduct a thorough functional check and to clean or disinfect the basic structure before using it again.

4. Cleaning, care and disinfection

The board itself is very resistant and can be cleaned with normal household cleaners.

Conventional cleaning: Use warm water and a mild cleaning agent for cleaning. If necessary, wipe the board using a damp cloth and then leave it to dry thoroughly.

Medical cleaning: The board can be effortlessly disinfected using an alcohol-based disinfectant solution. Always leave to dry thoroughly. The board is unsuitable for use in an autoclave due to the thermo-sensitive structure of the material.

5. Use / placement

1 Position the board to the side, next to the patient.

2 Ensure that the brake is on!

3 Tilt the patient to the side gently and slide the board slightly from under the patient.

4 Position yourself next to the transfer level and push the patient in the desired transfer direction.

5 To remove the board, reposition the patient in a short-term gentle lateral position.

6 The board can now be removed from under the patient.

6. Service life / life cycle

Board: If the product (board) is used in accordance with the safety regulations, the expected service life is 5 years. There is no restriction to this period. Depending on the frequency and intensity of use and the number of cleaning and disinfection procedures, this period can be significantly exceeded.

Disposable gliding cover: Opened rolls of disposable gliding cover should be used up within 6 months. Protecting the disposable gliding cover from dirt, heat, moisture and sunlight is essential, otherwise the gliding properties can degrade significantly (for more information, please take a look at the IFU for the disposable gliding cover). Unopened rolls have a shelf life of least 12 months after receiving the product.

7. Contraindications / risks of use

There are no known contraindications.

Users must be trained. If users are not trained or familiar with the correct handling of the hygiene board, the hygiene board may be lifted or pushed incorrectly. In addition, movements can be carried out too quickly and jerkily. This could lead to damage to the patient's and/or user's health. Observe the instructions for use under point 5 and the safety instructions under point 8.

It is the sole responsibility of the user to determine the suitability of the Transatlantic Hygieneboard for use. Transatlantic accepts no responsibility for improper use of Transaroll products and the gliding cover.

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WARNING signifies a possible hazard that could lead to personal injury or material damage.

- Defective products must no longer be used.
- Slip hazard! Ensure that the patient does not slide off the bed, recliner, etc. during transfer or repositioning.
- Risk of accident! Always make sure when using the Hygieneboard that the bed, recliner or any movable base is stable, i.e. that any brakes on it are locked.
- Ensure that the Hygieneboard is stable on both sides at the transfer levels. The bridging of free transfer levels with the Hygieneboard must not exceed 15 cm.
- It is the responsibility of the user/institution to evaluate if multiple users may be needed for the transfer or if additional safety measures must be taken.
- Never leave the patient unsupervised on the Transaroll Hygieneboard.
- The Transaroll Hygieneboard may only remain under the patient for a short time after the transfer.
- The Disposable Gliding Cover must not be reprocessed (see point 3).
- Ensure that the patient lies securely on the board and cannot slide off, especially when lifting the Transaroll Hygiene Board.
- Make sure that the transfers are performed by the users in a way that is back-friendly.
- Weight limits must be observed: Weight load for short-term lifting max. 300 kg.

Repositioning products are also subject to a natural ageing process. Inspect the product regularly and carefully. Ensure that the product is in flawless condition and that the material has not been damaged.

When using the Transatlantic Hygieneboard with the disposable gliding cover, please follow the separate instructions for use for the disposable gliding cover. The gliding cover is a disposable product and is not washable.

Transaroll transfer aids are subject to the highest quality requirements Any serious incidents must be reported to the manufacturer and the competent authority in the Member State where the user and/or patient is based.

9. Product marking /CE

The Transatlantic Hygieneboard is a Class I medical device and is compliant with EU Regulation 2017/745 of the European Parliament and of the Council dated 05.04.2017 regarding medical devices (MDR).

We are certified as a manufacturer and distributor of medical devices in accordance DIN EN ISO 13485.

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