Transaflow Multi-APS

24h safety

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
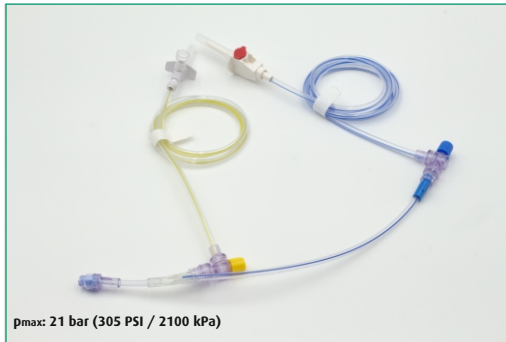
Multi-APS 24h safety

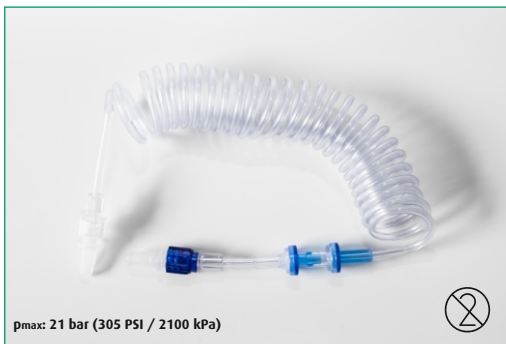- REF**
 2590401
 2590402
 2590403
 2590404
 2590405
 2590406

Patient lines Spiraled PWL with integrated germ barrier

- | | |
|------------|------------|
| REF | REF |
| 2580425 | 2580480 |
| 2580437 | 2580481 |
| 2580443 | 2580482 |
| 2580445 | 2580483 |

Patient lines Standard PWL with integrated germ barrier

- | | |
|------------|------------|
| REF | REF |
| 2580424 | 2580475 |
| 2580436 | 2580500 |
| 2580462 | 2580501 |
| 2580465 | 2580502 |
| 2580472 | 2580503 |
| 2580473 | 2580504 |
| 2580474 | |

ETHYLENE OXIDE STERILIZED

KEEP DRY

KEEP AWAY FROM SUNLIGHT

LATEX-FREE

DO NOT USE IF THE PACKAGE IS DAMAGED

STORAGE TEMPERATURE

DEHP-FREE

DO NOT RESTERILIZE

PYROGEN-FREE

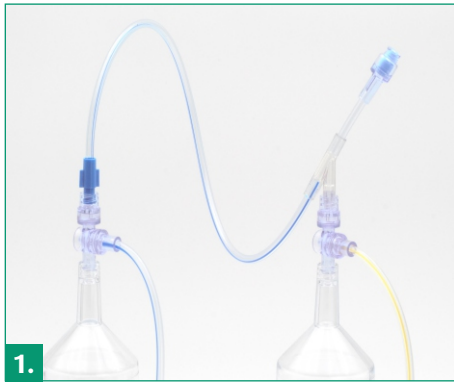
READ THE DOCUMENTS ENCLOSED

READ INSTRUCTIONS FOR USE ENCLOSED

2. Instructions

1.

Connect the Multi-APS 24h safety with the injector syringe:

yellow = contrast media
blue = saline (NaCl)

(Comply with the manufacturer specifications of the injector and syringes)

2.

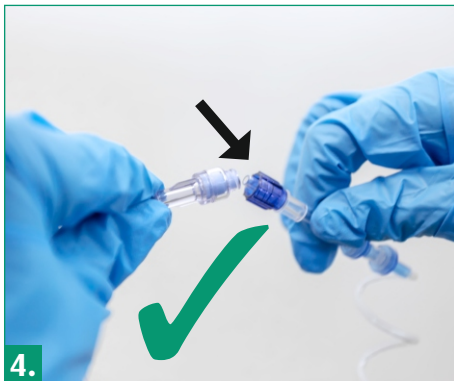
Connect the yellow coded line to the contrast media container.

3.


Connect the blue coded line with the saline container.

Fill the syringes with help of the injector.

4.

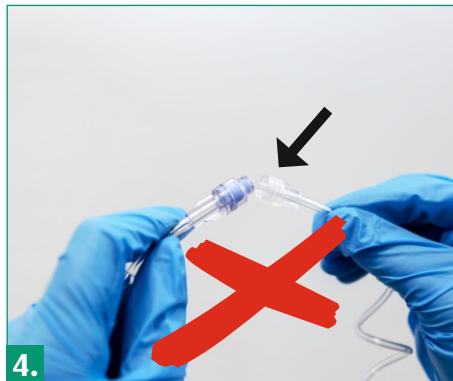Connect the blue coded end of the patient line with the blue safety adapter of the Multi-APS 24h safety.

4.

5.

Securely connect the transparent end of the the patient line with the catheter of the patient.

Every patient line must be flushed before usage!

Important!

- Right after connecting the application system to the injector, all lines must be flushed.
- Only apply pressure to the application system when the patient line is connected.
- Make sure that no pressure is applied when disconnecting the patient line from the application system.
- The patient line must be disposed after every use.
- Before connecting a new patient line to the application system, the safety adapter must be disinfected and dried according to the hygiene plan.
- When disconnecting the application system from the syringe a new application system must be used.
- The application system can be used for up to 24 hours.
- Touch the reservoir connections only at the grips.
- Do not re-use or recycle the application system.
- When reaching the max. pressure of the application system or the patient line, the application system and the patient line must be discarded.
- Note and comply with the manufacturer specifications of the saline and contrast media.
- The application system and the patient line are intended to be used by qualified personnel only.

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Transatlantic Handelsgesellschaft Stolpe & Co. mbH
Siemensstraße 21-23
61267 Neu-Anspach, Germany
Tel.: +49 (0)6081 94 30 50
Fax: +49 (0)6081 94 30 80
info@transat.de
www.transatlantic.de

🏭 Hersteller / Manufacturer:

CARDEA GmbH & Co. KG
Bruchweg 23
38822 Emersleben, Germany

Rev.: 01 / 15.11.2019